



Scan the QR code for more information

Hera II / 140

Bathing and toileting chair

User manual

Important information

This manual must be read before the product is used.

This manual is intended for technicians, therapists and other personnel, in connection with the use, servicing and maintenance of the Hera II bathing and toilet chair.

Date: 17-12-2019

Ver: 8

CONTENTS

1	PURPOSE AND SAFETY	4
1.1	Purpose	4
1.2	Safety instructions	5
1.3	Before use	5
1.4	Terminolgy	6
1.5	Transport and unpacking	6
2	USE AND ADJUSTMENT	7
2.1	Using the tilt function	7
2.2	Adjusting the accessories of Hera II	7
3	MAINTENANCE	9
3.1	Cleaning	9
3.2	Storage	9
4	CHECK AND WARRANTY	9
5	TECHNICAL DATA	9
6	ENVIRONMENT AND QUALITY	10
6.1	Disposal	10
6.2	Quality	10
6.3	Symbols	10
7	ACCESSORIES	11

1 PURPOSE AND SAFETY

This user manual describes the use of Hera II/140 bathing- and toileting chair, which have many possibilities for use, adjustments and describe the maintenance of the chair.

1.1 Purpose

The purpose of the Hera II bathing and toileting chair is to facilitate the work for the carer in connection with the user's needs for personal care and give the user an easier approach to personal hygiene, such as

- toileting, either in the bathroom or other suitable places in the home.
- when the user must have a shower or other kind of washing.

Remember: Always remember to take the user's personal needs and conditions into account.

Use:

Hera II/140 can be used for users with a maximum weight of 140 kg.

Hera II/140 is intended for indoor use under normal temperatures +20 to +40 C° and a relative humidity 40-90%.

Hera II/140 can be used on hospitals, nursing homes and in homecare.

Note! Never use the chair for any other purpose than originally intended.

1.2 Safety instructions



Important – warning!

This user instruction must be read thoroughly before the chair is taken into use.

- The maximum capacity is 140 kg. and this must **never** be exceeded.
- There must be no indication of wear and tear on the chair. NEVER use the chair if there is suspicion of any defect (abnormal sound, loose brackets etc.). In such cases it is important to find the cause and to take the chair out of service and contact your dealer or Winncare Nordic.
- Under special circumstances a build-up of static electricity can occur (e.g. using on a plastic floor or in very dry air). These conditions should be avoided.
- No smoking in the chair since the seat, back rest and neck support contain flammable materials.
- Brake the wheels if there is any danger of inadvertent movement i.e. on a ramp or other slope.
- When using the chair with other equipment ensure that no dangerous situations can arise. When using with a hoist ensure that sling straps cannot get tangled around the wheels.
- Make sure that the chair is correctly set for the user and either the user or carer is squeezed during use.
- It is the carer / institutions responsibility to ensure that using the chair is safe.

1.3 Before use

The carer/user is responsible for ensuring that a check is carried out BEFORE Hera II is used and have received instructions in its correct use.

Before use check following:

- 1) Ensure that there are no parts missing and the chair is suited the user.
- 2) Ensure that there are no signs of wear and tear or other defects.
- 3) Try the tilt function and ensure that the seat and the wheels are in order.
- 4) Ensure that accessories are in good working order.
- 5) If in any doubt concerning the use of Hera II contact Winncare Nordic or your dealer for advice and guidance.

1.4 Terminology

Hera II / 140



1.5 Transport and unpacking

Hera II/140 transported best by using the original packaging or on a pallet. We recommend that the packaging is kept if the chair for return shipment in case of repair or annual check.

Check that all the parts are present and undamaged.

Any suspicion of damage or shortage should be reported to your dealer or Winncare Nordic.

2 USE AND ADJUSTMENT

When the Hera II chair is going through corridors or over doorsteps (max. height 5 cm.) it is best to pull the chair, thus ensuring maximum stability.

Note! Ensure that the user keep his/her hands inside the armrest when going through doors and corridors. The user should sit calmly during this kind of movement.

2.1 Using the tilt function

Hera II chair can be adjusted from +5° (forwards) to -30° (backwards).

Activate the tilt function by pulling the cable lightly upward and tilt the chair to the desired position.

Note! This operation requires less effort if you push against one of the back legs with your foot whilst activating the handle.

2.2 Adjusting the accessories of Hera II

The chair should be adjusted to the individual user on delivery.

Setting the seat height:

Setting the seat height is carried out without tools.

It is not meant to be used on a daily basis.

- 1) Release the spring-loaded screw (A) on both sides and turn these so that they rest against the outer surface.
- 2) The seat can now be raised until the desired height is reached.
- 3) Turn the spring-loaded screw so that it engages again.



Warning: The spring-loaded screw **MUST** be engaged before using the chair again.

Foot supports:

- 1) Remove the clips and adjust the foot support to the required height.
- 2) Click the clips on.
- 3) If the foot supports can be angled, the support can be pulled out to the required angle and will lock automatically.



PU-Back rest:

The back rest will automatically angle itself according to the pressure of the user.

It can also be moved up or down to suit individual needs.

- 1) Loosen the screws on the brackets.
- 2) Adjust the back rest at the correct height and tightening the screws again.

Note! This does not apply to the fabric back.



Neck support:

- 1) Use the hand screw to adjust height and angle.



Armrest:

The armrests can be raised to allow sideways transfer.



Ensure that the user's arms/fingers cannot be trapped or squeezed when the armrests is raised up or lowered.

3 MAINTENANCE

3.1 Cleaning

It is recommended that Hera II chair is cleaned and disinfected regularly with a small quantity of standard household cleaning agent. The seat and accessories may be removed and washed down. All parts can be wiped with surgical spirit.

Note! If the chair is used for more than one person, it **MUST** be thoroughly cleaned.

Note! Strong chemicals or abrasive agents must not be used.

If the chair is used as a toilet chair, ensure that the bucket is correctly placed under the seat. The bucket and lid can be cleaned according to relevant regulations. The bucket and lid purchased by Winncare Nordic can be cleaned in a bedpan washer 85°C.

3.2 Storage

Hera II chair must be stored in a dry environment with relative humidity under 70%.

4 CHECK AND WARRANTY

Hera II chair must be serviced and thoroughly checked at least once a year by a qualified person. It is the legal owner of the product to ensure that the check is carried out.

If it is necessary to replace any parts due to wear or damage, it is recommended by Winncare Nordic, for safety reasons only to use original spare parts.

Warranty

There is a 10 years warranty on the frame and 2 years on other parts.

5 TECHNICAL DATA

	<u>Hera II /140</u>
Seat width	435 mm
Distance between arm rests	480 mm
Seat depth	420 mm
Seat height	530-610 mm
Total width	600 mm
Total height (top of handles)	1035-1115 mm
Angle of tilt	+5° to -30°
Weight of the chair	22 kg
Weight capacity (SWL)	140 kg

6 ENVIRONMENT AND QUALITY

Hera II chair has an expected life time of minimum 10 years under the assumption that all recommended service checks have been carried out and that the normal maintenance have been carried out.

6.1 Disposal



Disposal of Hera II can be made through Winncare Nordic. After lifetime use the chair can be returned to discard in an environmentally correct way either through reuse or recycling.

6.2 Quality

Winncare Nordic is certified according to the quality standard ISO 9001 and ISO 13485.

These certifications mean that the products and the company complies with the international standards for quality management and standards for traceability of medical products.

Hera II /140 is classified as a Medical device class 1 product.

6.3 Symbols



The product should be reused where possible



Refer to user instruction

SWL

Safe working load (user + accessories)


7 ACCESSORIES

Hera II:

- 041-00280 Hera II/140
041-00296 Hera II/140, only frame

Neck support

- 047-00310 Neck support

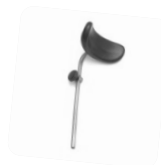


Back rest:

- 047-00203 Back rest, fabric
047-00210 PU back rest



Seat / lid:

- 047-00250 PU seat
047-00255 Seat with hygiene opening
047-00260 Lid for PU seat
047-00261 Lid for hygiene opening
047-00550 Gel cushion

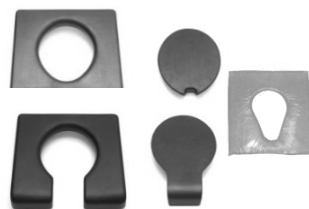

Foot / leg support:

- 047-00030 Foot rest, standard
047-00600 Foot rest adjustable
047-00610/611 Foot rest special, right / left



Side support:

- 047-00511 Side support



Bucket:

- 047-00902 Bucket with lid
047-00903 Bucket without lid

Other:

- 047-00350 Calf strap
047-00035 Width extension (arm rest)
047-00500 Seat extension
047-00509/510 Cross bar, right / left



A complete list of spare parts is available. Please contact Winncare Nordic.

Manufactuerer:
WinnCare Nordic Ltd.
Taarnborgvej 12 C
4220 Korsoer
Denmark

Phone: + 45 70 27 37 20
Fax: + 45 70 27 37 19
Email: info@winnCare.dk
www.winnCare.com

93/42/EØF CLASS I
EN/ISO 10535
EN 60601-1-2
UM-910-00765UK

