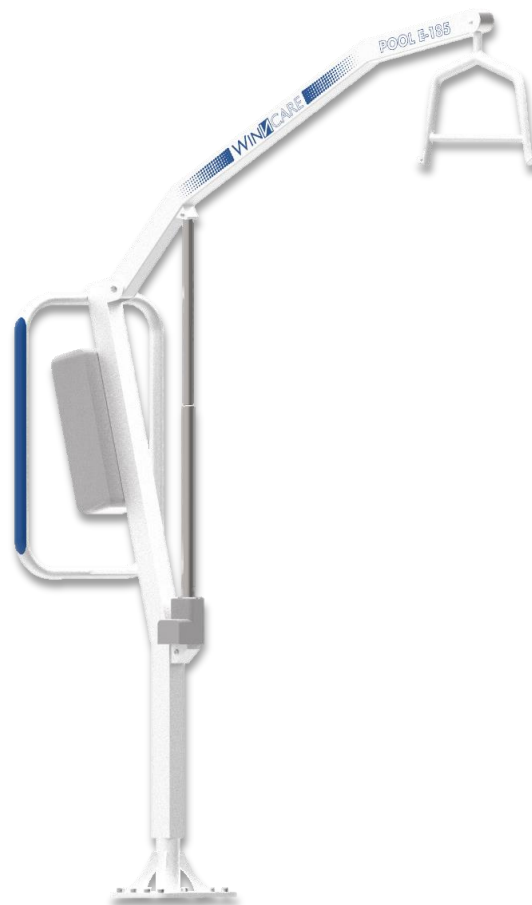
WINCARE

S p a i n

Pool E-185

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UK

Please read this instruction carefully before using your hoist; it contains useful information on safety, handling and maintenance.

Winnocare has wide experience in the manufacture hoists acquired throughout the years. The constant improvements introduced have bolstered out development as a Technical Aids manufacturing company, which is clearly reflected in our wide range of models and applications.

Winnocare hoists comply with the European Directive 2017/745 and so carry the CE mark. They have been tested at the Institute of Biomechanics of Valencia (IBV) and fulfil all the requirements laid down in the standard EN ISO 10535: "Hoists for the transfer of disabilities persons. Requirements and test methods".

These operating instructions has been written in accordance with the current desings of the products. All of them are constantly subject to a process of continuous improvement. Therefore, we reserve the right to modify any technical data that appears in these operating instructions without prior warning. All data, figures and measurements included in these operating instructions are approximate and must not be considered as strict technical specifications.

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UK

Every hoist is fully assembled in the factory to check that it works properly and, depending on the version, is dismantled or folded for packaging and transport.

Winnocare hoists are designed to facilitate assembly without use of any tool. Remove all parts of the hoist from the packaging (some of the parts are heavy and you may need someone else to help you).

1. Remove the pin on the hoist arm. Raise the arm until the holes at the end of the actuator is level with that of the securing plates.
2. Re-insert the pin.

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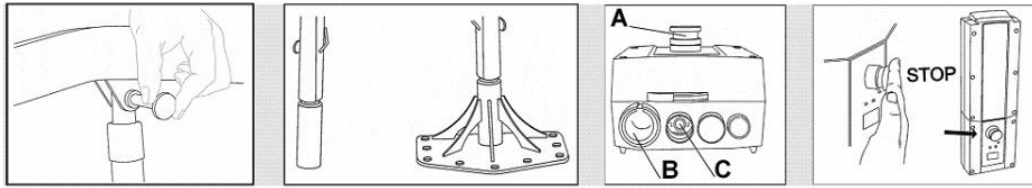
UK

1. The column is fully attached in its usage position.
2. Before using the hoist for the first time, it is advisable to charge the batteries for at least 5 hours.
3. Check that the connection of the actuator (C) and that of the handset (B) are firmly connected in the control box.
4. Press the up and down buttons to check that the actuator operates correctly.
5. Check that the red emergency stop button (A) is working correctly; if you press it, with the hoist moving up or down, it must stop.

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UK

Familiarise yourself with the hoist's devices and how they work before using it for real. Your hoist is designed for lifting patients, not for any other purpose. Check that the weight of the patient does not exceed the maximum weight that the hoist can lift or the sling bear.

Transfers must be done by 2 people: 1 person controlling the handset and the other one in the water to take care of the patient.

Check that the end of the actuator is fitted between the plate on the hoist arm and has its pin correctly inserted. Do not force the controls and devices of the hoist. All of them are easy to use and do not require excessive force to be applied.

Never leave a patient alone in a hoist.

Winnocare hoists have been designed and manufactured for use with Winnocare accessories; the accessories of other manufacturers have not been tested by Winnocare and so their use cannot be recommended. Do not use a worn sling.

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UK

The electric actuator has a safety system that disengages the motor if, in the downward movement of the hoist, it comes up against an obstacle that hinders it from descending. The hoist with dismantlable batteries have an emergency electrical lowering system, situated on the load display screen. This emergency electrical lowering system is operated by inserting the point of a ballpoint pen into that button, which makes the hoist arm descend if a fault has occurred in the handset (do not use a sharp object to press it).

When undertaking the raise-lower movement of the hoist keep the appropriate button pressed in, avoiding pressing the buttons repeatedly, as repeated start-up of the motor uses up more battery, discharging it.

The electrical equipment has an emergency stop button that disconnects the motor's battery automatically halting the movement of the hoist. To disengage the stop button, it must be turned. When the hoist is NOT in use, press the emergency stop button to prevent the batteries from running down. Keep the electrical equipment away from water and other liquids to protect it from possible splashes that may affect its internal circuits causing serious damage.

The batteries must be replaced after 4 years and perhaps before, depending on the type of use. Heavy and frequent unloading reduce the life of the battery. For the optimum life of the battery recharge it as often as possible. Recharging time is around 5 to 8 hours. If the hoist is used daily, it is a good idea to recharge the battery at night. The control box has an authentic device that disconnects it from the charger when it is fully charged. We recommend not undertaking 20 consecutive transfers without recharging up the batteries. Batteries of equipment that are not habitually used must be recharged at least every 3 months; otherwise they could be damaged as a consequence of their own self-discharge.

To charge, you must check that the connections are correctly attached.

Remember: the battery is the only component of the electrical equipment that requires continuous maintenance. If you do not follow the advice above, the battery may discharge to a level that does not allow it to recover its full charge or may even render it useless.

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UK

To recharge the battery you must use an additional external charger. To do so, extract the batteries from the hoist by raising the lever on top of them and pull the battery box upwards and outwards. Place the battery box in the external charger making sure you hear the "click" that confirms they have been fitted properly. Connect the charge to the main supply and then follow the same instructions as above.

The electrical equipment has a screen on which the state of the battery charge is displayed. We recommend that the charge of the battery in use does not fall below 50%, the moment when it is essential to charge it.

To recharge the battery you must connect the cable supplied and connect it to the main supply (220V). A green light will come on indicating that the charger is connected to the main supply and a yellow light will remain lit up until the battery is charged. The batteries can be recharged using an additional external charger (accessory). To do so, extract the batteries from the hoist by raising the lever on the top of them and pull the battery box upwards and outwards.

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UK

For all its electrical hoist equipment, WinnCare uses the Linak company, a world leader in manufacturing this type of equipment.

The electrical equipment complies with the following standards:

IEC 60601-1 Medical electrical equipment.

Part 1: General requirements for basic safety and essential performance.

IEC 60601-1-2 Medical electrical equipment.

Part 1-2: General requirements for safety collateral standard: electromagnetic compatibility. Requirements and tests.

The maximum work cycle of the electrical equipment is 10% or 2 minutes of continuous use followed by 18 minutes rest. If this work cycle is exceeded there is a risk that the equipment will overheat and cause a break-down.

Operating environment of the electrical equipment: from + 5° C to + 40° C.

Actuator 24V continuous current. Batteries 1 x 24V / 2.9Ah sealed rechargeable batteries of the lead-acid type.

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All WinnCare hoists have been tested according to EN ISO 10535, and have a minimum lifespan **of 6 years**. In order to prolong the useful life of the hoist, it is necessary to continue with the annual revisions by our official technical service.

The following table indicates the estimated life of the motor / actuator depending on the number of elevations per day.

Elevation X day	Years life
30	1
15	2
10	3
7-8	4
6	5
5	6
4-5	7
3-4	8
2-3	9
1-2	10

The above check list is a guide for LINAK actuators used in conditions of maximum load according with LINAK specifications and in line with the standard EN10535 Patient Hoist.

If the pattern of use, load and run differs from LINAK specifications, this table should be adjusted to suit the application.

The LINAK specifications mentioned can be found on the actuator label, or if it is no there, they can be consulted in the corresponding data sheets at www.linak.com

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UK

The hoist must be cleaned using a damp cloth with a product for cleaning furniture or similar that does not contain solvents that could damage the coating of the hoist. Do not allow the electrical elements of the hoist to come into contact with water.

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UK

The hoist is manufactured with stainless steel profiles with epoxy coating. Therefore, to dispose of it you must follow recycling regulations or local recommendations on disposal.

Rechargeable batteries contain lead (Pb) must be recycled so they should be disposed them of in containers destined for recycling cars batteries or at appropriate waste management sites.

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UK

If the problems cannot be solved through these suggested recommendations, contact he supplier.

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UK

All hoists have been rigorously inspected; on their labels there is a lot and series number that lists that hoist with the materials, equipment, and components employed and the work processes undertaken for their manufacturing assembly and inspection. The guarantee is for 5 years. 1 year for the battery, under normal usage conditions, and respecting the recommendation on use and maintenance and not changing or manipulating any of their components. Any manipulation or alteration, especially of the electrical equipment renders this guarantee null and void. In the event of defect or malfunctioning of the hoist or any of its components, **contact the distributor where you purchased it**, presenting it together with the invoice or purchase receipt, so that it may be inspected so that the manufacturer can decide whether to repair it or replace it. Winnocare accepts no responsibility for imperfections caused by bad usage or as a consequence of not having followed the recommendations given in these operating instructions.

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


IDENTIFICATION OF THE MEDICAL EQUIPMENT		ESTABLISHMENT		
Category:		Department/Location:		
Brand/model/type				
Serial number:	Inventory number:			
Date of manufacture:				
VISUAL INSPECTION (according to ISO 10535)		NA	C	NC
Availability and condition of the instruction manual, loading plate, identification plate/labelling/printing				
Good general condition (structure, base, articulations, welds, attachments, etc...). Absence of rusting				
Good general cleanliness				
Good stability of the hoist and good operation of the base opening/closing				
Checking of the tightness of screws (suspension pin on the lifting arm, weighing module attachments, etc.), absence of play.				
Good condition of slings and cradle or hammock, including the attachment system				
Visually check that the pin is present in the hole of the safety nut. Visually check that the pin is depressed approximately 2mm from the outside of the edge of the safety nut. The pin must not protrude.				
				
Good rotation of the gripping bar and good condition of the hooks				
Good operation of the wheels and brakes (swivelling, running, locking)				
Good lubrication of articulations (absence of noise nuisance, squeaking and hard points, etc.)				
Good condition of electric cables, control boxes, connectors and actuators (jacks, motors, etc.)				
Good condition of controls and indicator lights				
SAFETY INSPECTION (according to ISO 10535)		NA	C	NC
Locking out of operational functions and correct operation of visual and audible alarms				
Emergency stop operates correctly				
Battery and charger operate correctly				
Obstacle stop operates correctly				
Correct operation of the emergency descent				
Test at maximum load (1 complete cycle)				
INSPECTION OF THE AMPLITUDE OF MOVEMENTS		NA	C	NC
Satisfactory operation of the jack (elevation/descent)				
Maximum and minimum height (according to the manufacturers specifications)				
ELECTRICAL SAFETY (according to 60601-1)		NA	C	NC
Tester model:	Serial number:			
Continuity :				
Breakdown:				
COMMENTS:				
Dated:	Inspector's signature and stamp	Date of the next inspection		
Name:				

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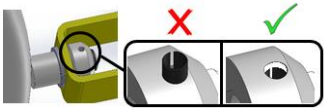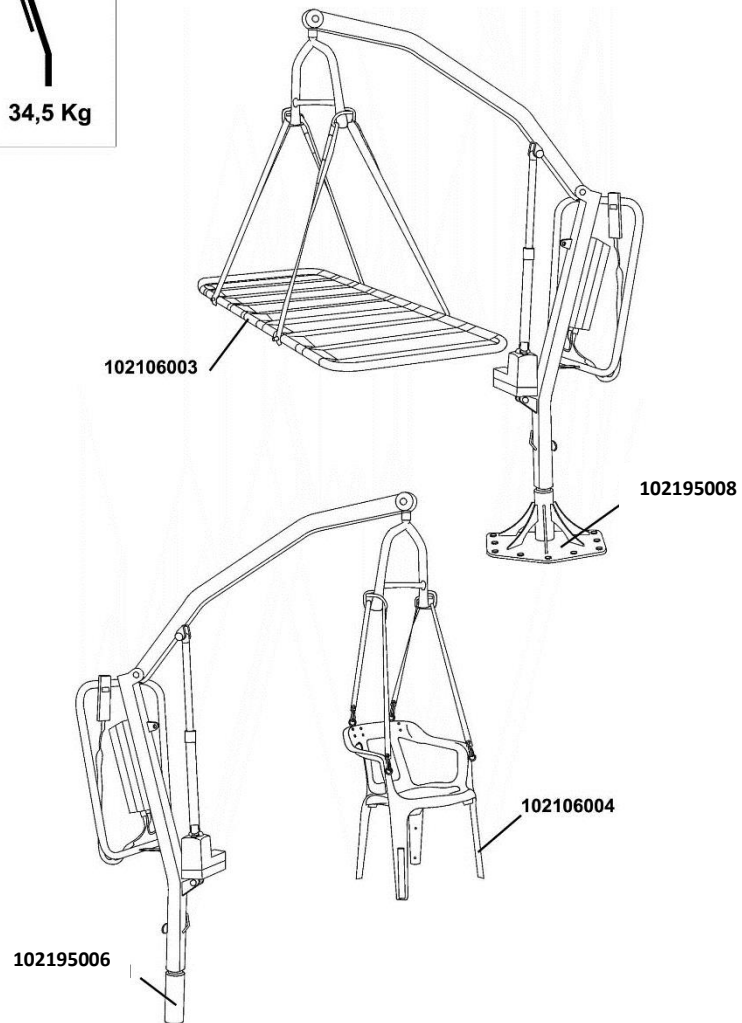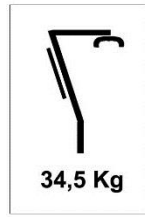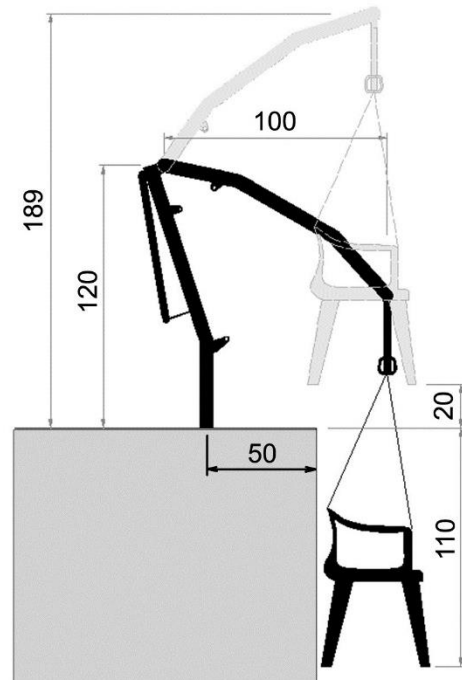
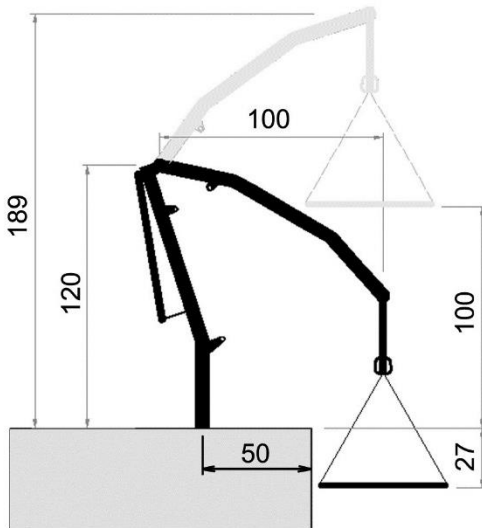
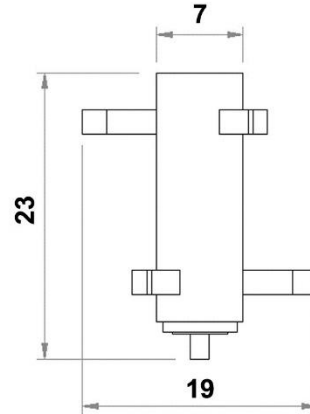
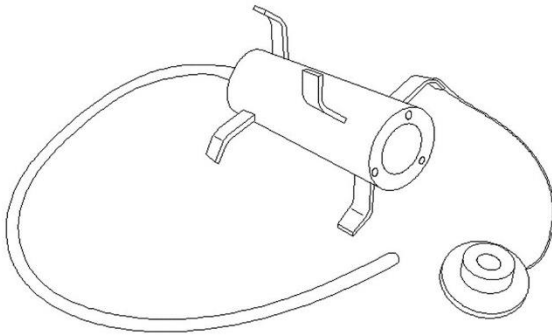
UK

Deck anchor for securing to cement base

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F

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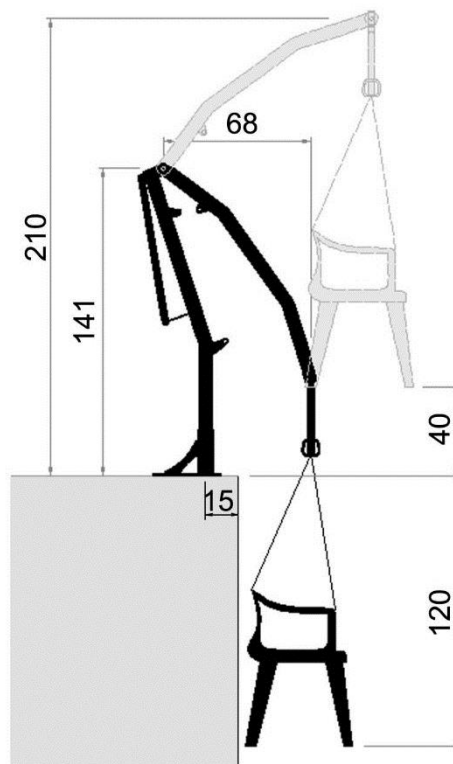
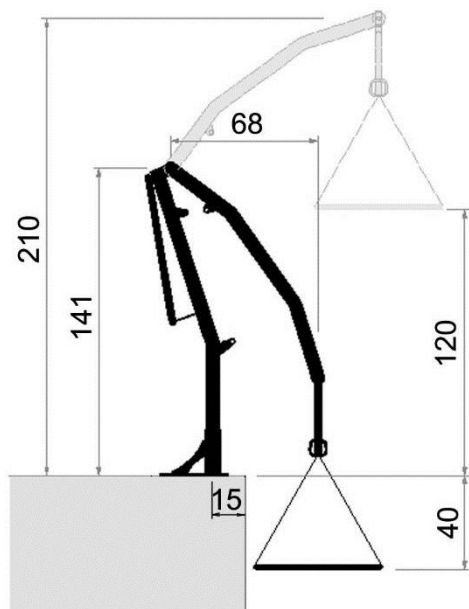
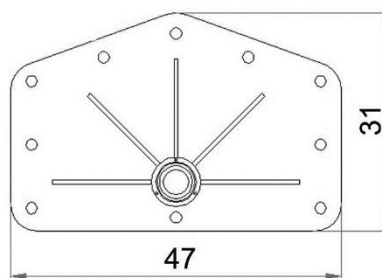
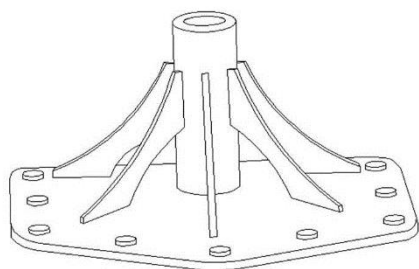
UK

Deck anchor for securing with screws

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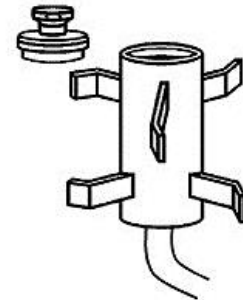
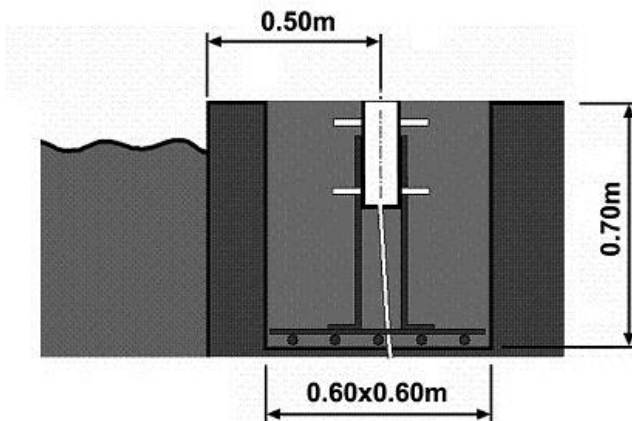
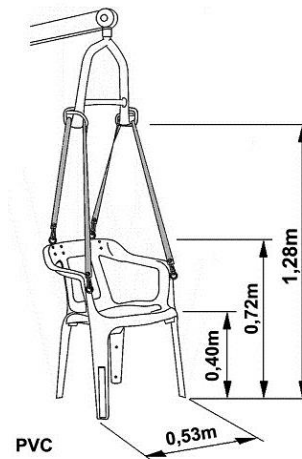
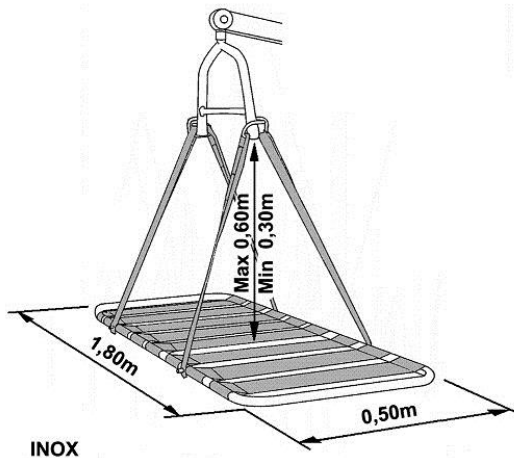
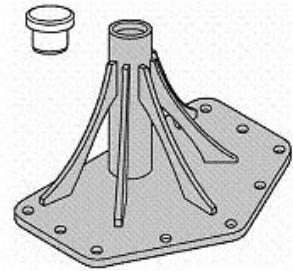
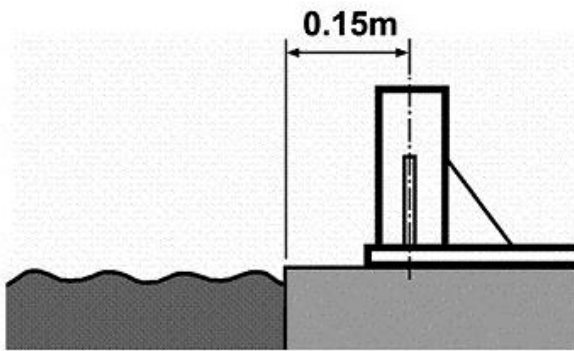
UK

To fix this deck anchor it will be necessary to have a sufficient quantity of concrete to realice an anchor of 60x60cm with a depth of 70cm, arming it with a gauge Steel as the work of cementing.

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UK

You must find a place where the ground is totally flat and where the ground is made of concrete (concrete HA-25 of 250 kg). It is very important to ensure that the deck anchor is completely perpendicular at the horizontal of the ground; otherwise, during the assembly of the hoist, the deck anchor will not rotate correctly.

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